NOR**MYKE KONTAKTLINSER**

Gode råd ved bruk

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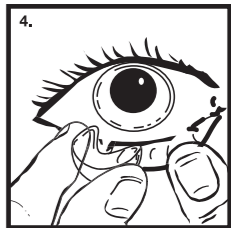
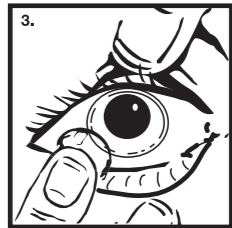
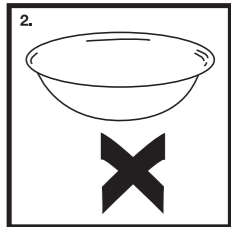
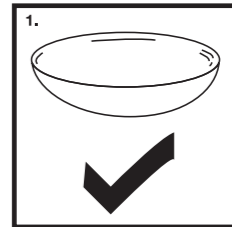
SOFT CONTACT LENSES WEAR INSTRUCTIONS FOR USE

GB**US**

SOFT CONTACT LENSES

Instructions for Use

A Guide to Trouble Free Soft Contact Lens Wear

This leaflet contains important information, please read carefully and retain for future reference. Contact lenses should always be fitted by a contact lens practitioner. It is essential that you follow your practitioner's directions and all labelling instructions for the proper use of your contact lenses and lens care products.

HANDLING YOUR LENSES

Your contact lens practitioner will provide you with detailed instructions on lens handling, insertion and removal. The lenses are packaged in a bottle which contains physiological salt solution. Always check that packaging of new lenses is intact. **Use only if the foil blister packs/vials are intact.** Always wash, rinse and dry your hands thoroughly before handling your lenses. Do not insert if the lens appear damaged. **Lenses must be removed before sleeping.** After wear, lenses must be removed from the eye, cleaned and disinfected using contact lens care products recommended by your contact lens practitioner, prior to placing back in the eye.

LENS INSERTION

Do not apply make-up until you have inserted your lenses. To avoid mixing up your lenses, always work with the same lens first. Make sure the lens is not inside out before insertion (see figures 1 & 2). Place the lens on your index finger to verify shape. Using your other hand's index finger and second finger hold up the upper lid and hold down the lower lid. Place the lens on the eye (see figure 3). Repeat for the other eye.

LENS REMOVAL

Look up and hold down the lower lid. Using your index fingertip and thumb tip to gently pinch the lens off the eye (see figure 4). Repeat for the other eye.

LENS CARE DIRECTIONS

Proper care of your contact lenses is essential. Always follow exactly the instructions for your lens care system. Disinfect your lenses each time they are removed to destroy harmful bacteria and to ensure safe, comfortable contact lens wear. Do not change from the solutions recommended by your contact lens practitioner.

PRECAUTIONS

Always remove your lenses before going to sleep.

Do not use any in-eye medication without consulting your practitioner.

Do not rinse your lenses or soaking case in tap water.

Do not allow hairspray or other cosmetic aerosols to come into contact with your lenses.

Do not change your brand of care products without consulting your contact lens practitioner.

Do not wear your lenses more than the period recommended by your specialist and respect the advised renewal deadline.

Do not wear your lenses if you are exposed to noxious or irritating fumes.

Visit your contact lens practitioner on a regular basis.

If you note the following symptoms: an irritated or red eye, feelings of discomfort or watering... remove the lens and clean it. If the discomfort continues, consult your specialist.

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