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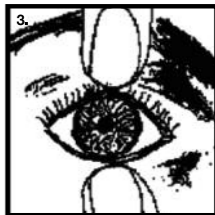
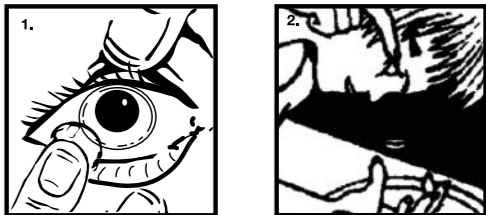
RIGID CONTACT LENSES WEAR INSTRUCTIONS FOR USE

This leaflet contains important information, please read carefully and retain for future reference. Contact lenses should always be fitted by a contact lens practitioner. It is essential that you follow your practitioner's directions and all labelling instructions for the proper use of your contact lenses and lens care products.

HANDLING YOUR LENSES

Your contact lens practitioner will provide you with detailed instructions on lens handling, insertion and removal. The lenses are packaged in a case which contains a decontaminating solution. Always check that packaging of new lenses is intact. **Use only if the foil blister packs/vials are intact.** Always wash, rinse and dry your hands thoroughly before handling your lenses. Do not insert if the lens appear damaged. **Lenses must be removed before sleeping.** After wear, lenses must be removed from the eye, cleaned and disinfected using contact lens care products recommended by your contact lens practitioner, prior to placing back in the eye.

PUTTING IN YOUR CONTACT LENSES

Do not put make up on before putting your contact lenses in. To avoid mixing up your contact lenses, always begin with the same contact lens. Place the contact lens on your index finger. Open the eyelids using the middle finger next to the index finger on which the contact lens is placed and the thumb of the other hand. Place the contact lens on the eye (figure 1). Repeat this procedure for the other eye.

TAKING OUT YOUR CONTACT LENSES

1st method: lean your head over a mirror placed flat on a table. Place your index finger in the angle of your eyelids and tug back gently whilst keeping your eyes wide open (figure 2): the lens will come unstuck and will fall on the mirror.

2nd method: place your right index finger on the lower eyelid at the base of the eyelashes, and your left index finger on the upper eyelid. Open your eyes wide and move your fingers closer together whilst applying light pressure to make the contact lens come unstuck (figure 3).

LENS CARE DIRECTIONS

Proper care of your contact lenses is essential. Always follow exactly the instructions for your lens care system. Disinfect your lenses each time they are removed to destroy harmful bacteria and to ensure safe, comfortable contact lens wear. Do not change from the solutions recommended by your contact lens practitioner.

PRECAUTIONS

Always remove your lenses before going to sleep.

Do not use any in-eye medication without consulting your practitioner.

Do not rinse your lenses or soaking case in tap water.

Do not allow hairspray or other cosmetic aerosols to come into contact with your lenses.

Do not change your brand of care products without consulting your contact lens practitioner.

Do not wear your lenses more than the period recommended by your specialist and respect the advised renewal deadline.

Do not wear your lenses if you are exposed to noxious or irritating fumes.

Visit your contact lens practitioner on a regular basis. If you note the following symptoms: an irritated or red eye, feelings of discomfort or watering... remove the lens and clean it. If the discomfort continues, consult your specialist.

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