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## ORTHOKERATOLOGY CONTACT LENSES WEAR INSTRUCTIONS FOR USE

**GB****US**

Instructions for Use

This leaflet contains important information, please read carefully and retain for future reference. Contact lenses should always be fitted by a contact lens practitioner. It is essential that you follow your practitioner's directions and all labelling instructions for the proper use of your contact lenses and lens care products.

## HANDLING YOUR LENSES

Your contact lens practitioner will provide you with detailed instructions on lens handling, insertion and removal. The lenses are packaged in a case which contains a decontaminating solution. Always check that packaging of new lenses is intact. **Use only if the foil blister packs/vials are intact.** Always wash, rinse and dry your hands thoroughly before handling your lenses. Do not insert if the lens appear damaged. After wear, lenses must be removed from the eye, cleaned and disinfected using contact lens care products recommended by your contact lens practitioner, prior to placing back in the eye.

## PUTTING IN YOUR CONTACT LENSES

To avoid mixing up your contact lenses, always begin with the same contact lens. Place it on your index finger, fill it with a comfort solution (figure 1) and lean your head back. Use the index finger and the thumb of the other hand to open the eyelids. Place the contact lens on the eye (figure 2). Repeat this procedure for the other eye.

## TAKING OUT YOUR CONTACT LENSES

**20 minutes before removal of the lens add a drop of the comfort solution.**

**Lean your head over a mirror placed flat on a table.** With the eyes wide open, place the index finger of one hand on the upper eyelid at the edge of the eyelashes and the index finger of the other hand on the lower eyelid at the edge of the eyelashes, move the two index fingers closer together whilst applying light pressure to make the contact lens come unstuck (figure 3).

## LENS CARE DIRECTIONS

Proper care of your contact lenses is essential. Always follow exactly the instructions for your lens care system. Disinfect your lenses each time they are removed to destroy harmful bacteria and to ensure safe, comfortable contact lens wear. Do not change from the solutions recommended by your contact lens practitioner.

## PRECAUTIONS

Do not use any in-eye medication without consulting your practitioner.

Do not rinse your lenses or soaking case in tap water.

Do not allow hairspray or other cosmetic aerosols to come into contact with your lenses.

Do not wear your lenses when you are exposed to dangerous or irritating fumes.

Do not change your brand of care products without consulting your contact lens practitioner.

Do not wear your lenses more than the period recommended by your specialist and respect the advised renewal deadline.

Visit your contact lens practitioner on a regular basis. If you note the following symptoms: an irritated or red eye, feelings of discomfort or watering... remove the lens and clean it. If the discomfort continues, consult your specialist.

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