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Always wash and dry your hands before handling your lenses.

Do not use if the packaging is not sealed.

**Contact lenses are not to be shared.**

## LENS INSERTION

1. Before insertion, ensure the lens is not inside out. Place on your index finger to verify the shape (fig. 1 & 2).
2. Using middle finger, hold your lower lid down. With the index finger of the other hand, raise upper lid. Place the lens onto the eye (fig. 3). Repeat for the other eye.

## LENS REMOVAL

Separate the lids with the middle finger of each hand. Look up and with the index finger of the lower hand slide the lens down onto the white of the eye. Pinch the lens and lift out with the thumb and index finger (Fig. 4). Repeat for the other eye.

## DISINFECTION/SOAKING

For lenses worn more than once clean and disinfect them following strictly the instructions on your solution bottle or as directed by your eye care practitioner.

N.B. Daily disposable lenses are intended to be worn only once and then discarded. They are not designed to be cleaned or disinfected and worn more than one time.

## IF THE LENSES DRY OUT

Re-wet with disinfecting, soaking or rinsing solution then follow the Disinfection/Soaking step above.

## PRECAUTIONS

1. For all types of lenses, your eye care practitioner will recommend a lens wearing, cleaning and replacement regime. Do not vary these. If in doubt seek the advice of your eye care practitioner.
2. For lenses worn more than once, always clean and disinfect your lenses as instructed, after lens removal.
3. If your eyes become red or irritated or you experience any pain remove your lenses immediately and consult your eye care practitioner.
4. Avoid contaminating lenses with make-up, aerosol sprays or wearing them near noxious fumes.
5. Replace damaged, contaminated or scratched lenses immediately.
6. Avoid long nails, which harbour bacteria and could damage the eye or lens. A scratched eye is vulnerable to infection.
7. Keep your lenses out of the reach of children.
8. Only use lenses for water sports if watertight goggles are worn.

## WARNINGS

1. DO NOT sleep in your lenses unless your eye care practitioner has advised it is safe to do so.
2. DO NOT wear your lenses beyond the period recommended by your eye care practitioner.
3. DO NOT use household products (e.g. disinfectants) on your lenses
4. DO NOT wear your lenses when using eye drops (unless advised to do so by a medical practitioner).
5. DO NOT use saliva to wet your lenses.
6. DO NOT rinse your lenses or lens case with tap water.
7. The lenses could be uncomfortable during the performance of aquatic sports or when the lenses are exposed to toxic or irritant liquids or a gas as well as during dangerous situations associated with professional activity.

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