



Medmont 6 & 7

Topographies import



Medmont 6 and 7

- Open the chosen topography and click on « *Analyse* » puis « *Raw Export* »
- Save the datas into the chosen folder
- In « File name" name the imported topographies, for ex: « initial topo OD" »
- Click on « Save »

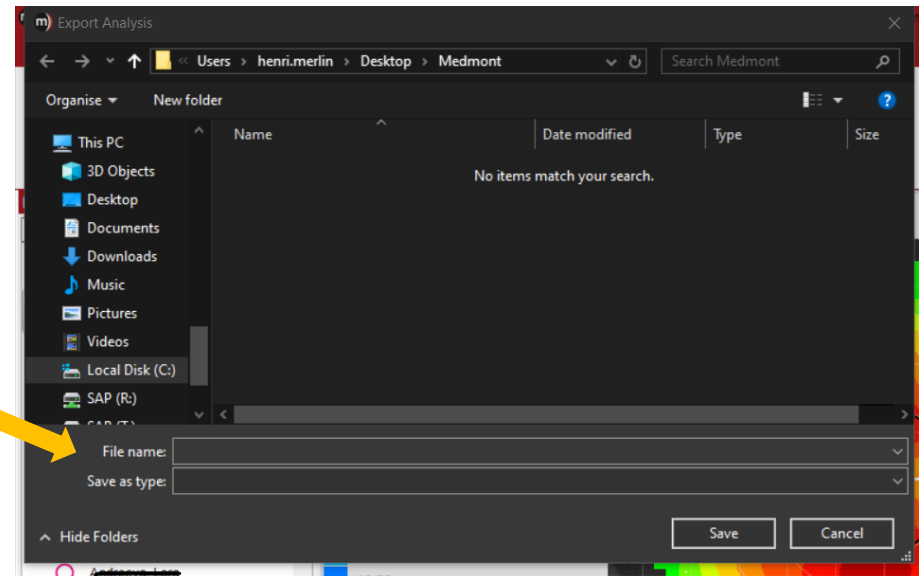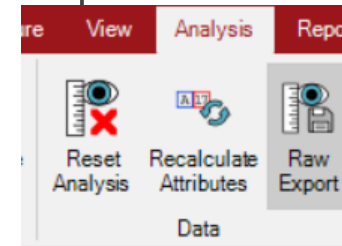

Medmont 6 and 7

Keratometry

Display unit
☒ Millimeters ☐ Diopters

Topographer model used
-

K flat	Exc flat	Axis (f)
	0,5	
K tight	Exc tight	DIV at 30° (Øcorneal)
	0,5	

K medium : 0.00

Topography





Save & import a topography for the Right Eye ×

Select a topographer*

Medmont 6 / Medmont 7 ▼

 Attention, in the case of a pathological cornea:
Do not use the data cleaning function.

☐ Clean topography ☐ Import pivot format

For the Medmont 6 and 7, three files are expected .tgl, .hgt and .dst, named : 'name_firstname_eye_date'

Select the file(s)
or drag and drop it here

Annuler **Import** 

- In the patient's C&F folder, click on the symbol : 
- Drag the previously saved files into the frame that is displayed and click on

Importer 