


Oculus Pentacam


Topographies import



Oculus : Pentacam

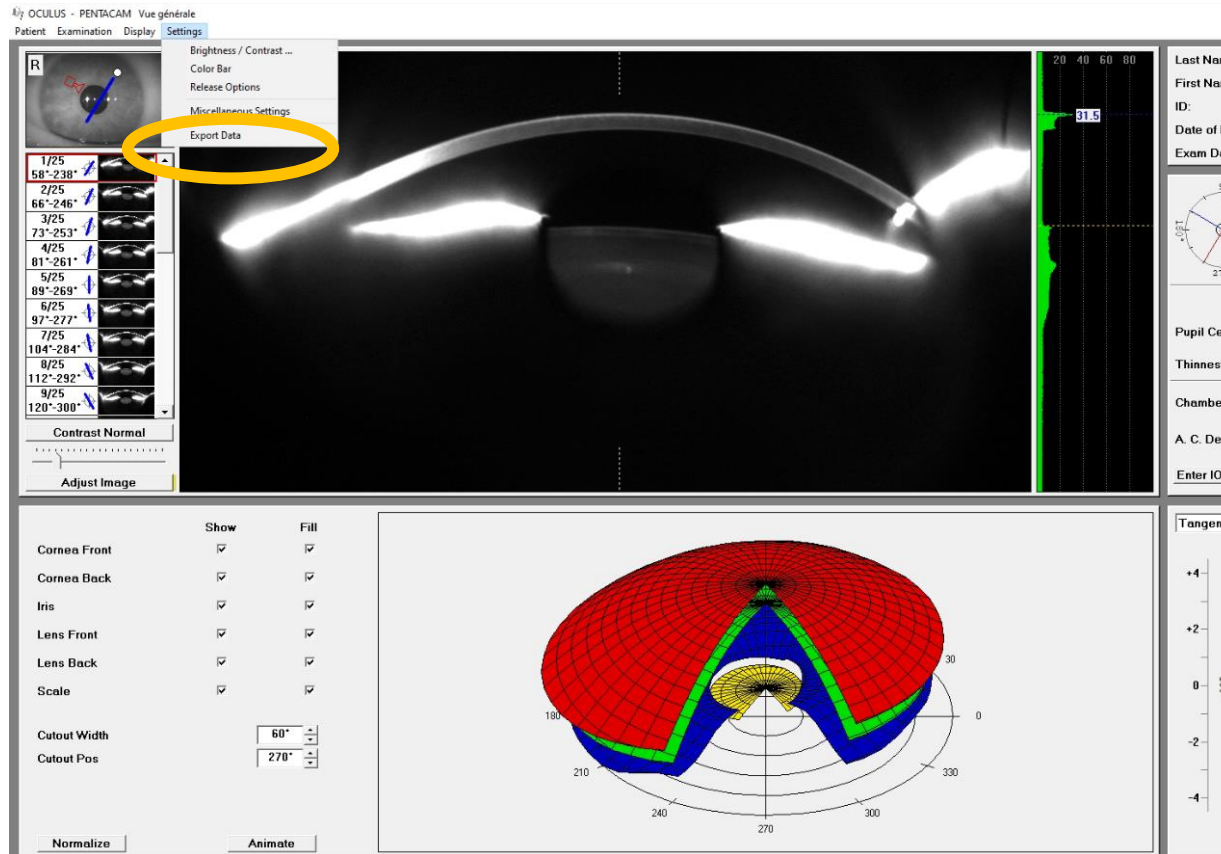


In « Settings », click on « Export datas », and the you'll find the files here

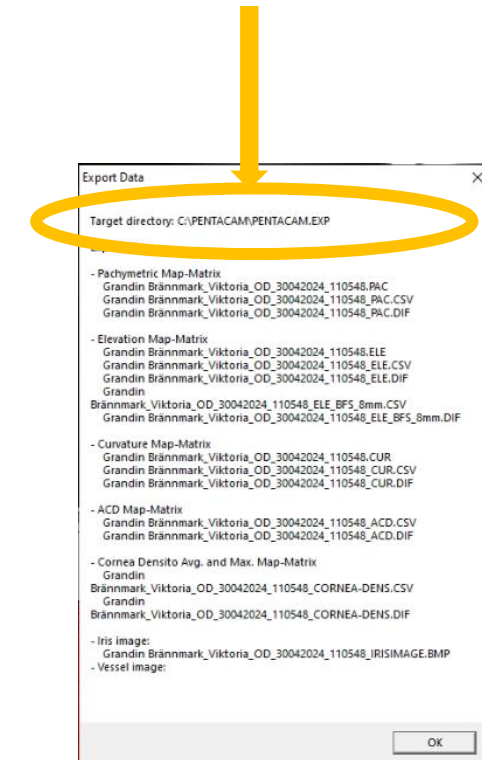

Oculus : Pentacam

- The .CUR file can appear like this :

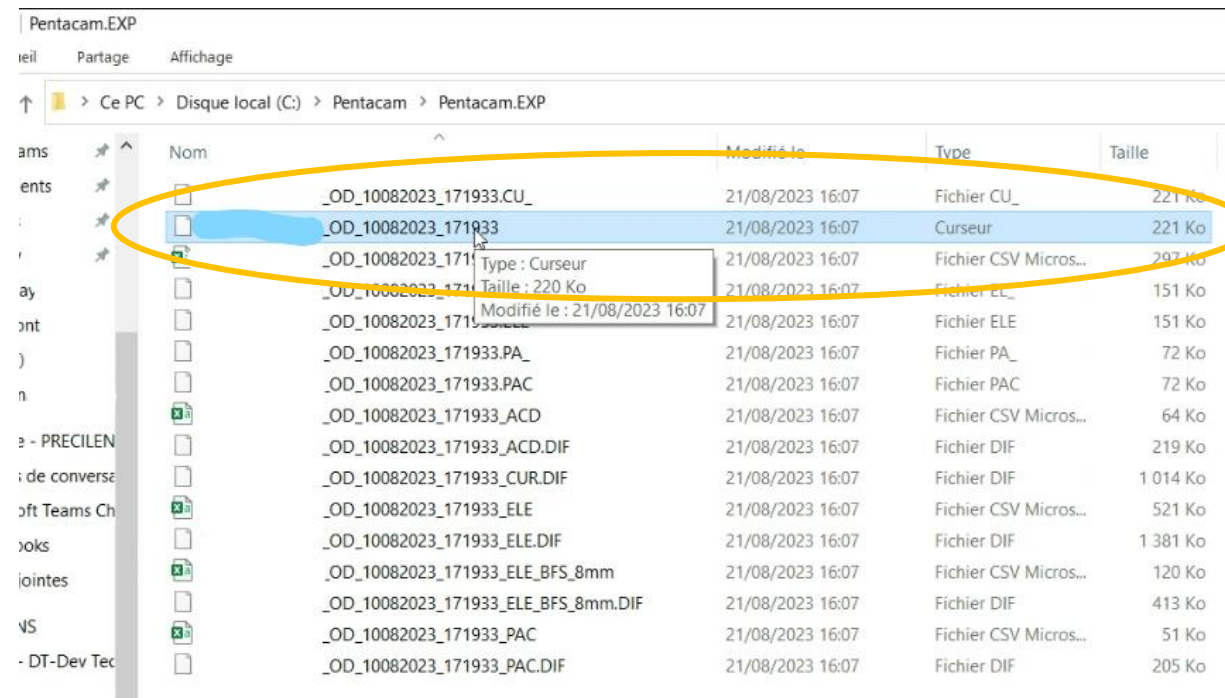

Pentacam.EXP

Partage Affichage

> Ce PC > Disque local (C:) > Pentacam > Pentacam.EXP

Nom	Modifié le	Type	Taille
_OD_10082023_171933.CU_	21/08/2023 16:07	Fichier CU_	221 Ko
 _OD_10082023_171933	21/08/2023 16:07	Curseur	221 Ko
 _OD_10082023_171933	21/08/2023 16:07	Fichier CSV Micros...	297 Ko
 _OD_10082023_171933	21/08/2023 16:07	Fichier ELE	151 Ko
 _OD_10082023_171933	21/08/2023 16:07	Fichier ELE	151 Ko
 _OD_10082023_171933.PA_	21/08/2023 16:07	Fichier PA_	72 Ko
 _OD_10082023_171933.PAC	21/08/2023 16:07	Fichier PAC	72 Ko
 _OD_10082023_171933_ACD	21/08/2023 16:07	Fichier CSV Micros...	64 Ko
 _OD_10082023_171933_ACD.DIF	21/08/2023 16:07	Fichier DIF	219 Ko
 _OD_10082023_171933_CUR.DIF	21/08/2023 16:07	Fichier DIF	1 014 Ko
 _OD_10082023_171933_ELE	21/08/2023 16:07	Fichier CSV Micros...	521 Ko
 _OD_10082023_171933_ELE.DIF	21/08/2023 16:07	Fichier DIF	1 381 Ko
 _OD_10082023_171933_ELE_BFS_8mm	21/08/2023 16:07	Fichier CSV Micros...	120 Ko
 _OD_10082023_171933_ELE_BFS_8mm.DIF	21/08/2023 16:07	Fichier DIF	413 Ko
 _OD_10082023_171933_PAC	21/08/2023 16:07	Fichier CSV Micros...	51 Ko
 _OD_10082023_171933_PAC.DIF	21/08/2023 16:07	Fichier DIF	205 Ko

Type : Curseur
Taille : 220 Ko
Modifié le : 21/08/2023 16:07

Oculus : Pentacam

Keratometry

Display unit
☒ Millimeters ☐ Diopters


Topographer model used
-

K flat	Exc flat	Axis (f)
	0,5	

K tight	Exc tight	DIV at 30° (Øcorneal)
	0,5	

K medium : 0.00

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Topography





Save & import a topography for the Right Eye

Select a topographer*

Oculus Pentacam

ⓘ Attention, in the case of a pathological cornea:
Do not use the data cleaning function.

☐ Clean topography ☐ Import pivot format

For the Oculus Pentacam, two files are expected : *.CUR
and *.ELE

Select the file(s)

or drag and drop it here

Annuler

Import 

- In the patient's C&F folder, click on the symbol : 
- Drag the previously saved files into the frame that is displayed and click on

Importer 