Customer name:
Oté Pharma Sol B.V.

CCL reference:
218057

Customer reference:
01REFM009-10

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Date / time:
08-Aug-22
13:42:29

Customer support:
Carin van der Steen

We request you to verify this PDF-proof on text, illustrations, lay-out and colour: The colours in this PDF give only an indication of the final result, depending the printing technique and the to be printed material. We emphasize that you, as client, are responsible for the correctness of the delivered artwork, the check on text, adoptions and other alterations. We do not accept any responsibility for printproofs with incorrect approvals.

Approved by customer	Approved by CCL
_____	OKE MODEL
Date:	Date:
_____ - 202_	_____ - 202_

EN All-In-One-Solution VC
Cleaning, disinfecting, inserting and storage.
For hard contact lenses.

Composition:
PHMB 0.0002%, POLYSORBATE 80, SODIUM CHLORIDE, EDTA, METHYLHYDROXYPROPYLCELLULOSE, BORAX/BORIC ACID, PURIFIED WATER.

Precautions and instructions for use
Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel.
Do not use when tamper-evident seal is broken before the first time use. Unscrewing the cap from the bottle will break the tamper-evident seal.
Immediately after removal of the lenses, clean them with 8 drops contact lens solution.
Clean the lenses by carefully rubbing them with your index finger in the palm of your other hand for 30 seconds.
Rinse the lenses with contact lens solution.
To disinfect the lenses, place them in the lens case filled with fresh solution and leave to stand for at least 6 hours.
Insert the lenses into your eyes.
Rinse the lens case with contact lens solution and allow the lens case to dry in the air.
Repeat disinfection steps when the lenses are stored unused in the lens case for a period longer than 7 days.
Do not touch the dropper tip of the bottle as this may contaminate the solution.
Close the bottle lightly immediately after use.
Replace your lens case frequently to prevent infection due to contamination.

Storage conditions
Use within 4 months after opening.
Keep out of reach of children. (See image 1)
Keep the solution in its original packaging.

Warnings and contra-indications
Do not use if you are allergic to one of the ingredients.
Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor.
If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lens solution.
Do not use when the packaging is damaged. (See image 2)
Do not use after the expiry date.
Do not change your contact lens care system without consulting your contact lens specialist.
Always replace any remaining solution in the lens case with fresh solution.
Never use tap water as an alternative for contact lens solution.
Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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Date / time:
08-Aug-22
13:42:29

Customer support:
Carin van der Steen

We request you to verify this PDF-proof on text, illustrations, lay-out and colour. The colours in this PDF give only an indication of the final result, depending the printing technique and the to be printed material. We emphasize that you, as client, are responsible for the correctness of the delivered artwork, the check on text, adoptions and other alterations. We do not accept any responsibility for printproofs with incorrect approvals.

Approved by customer	Approved by CCL
_____	_____
_____	_____
Date:	Date:
_____ - 202_	_____ - 202_

CCL Label Oss B.V.
IJsselstraat 15
P.O. Box 104
5340 AC Oss
The Netherlands
customerservice@cclind.com

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