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GB Recommendations for use

Guide to the use of your hard lenses

This leaflet contains important information; read it and keep it for future reference. Lenses are medical devices and must always be adjusted by a contact lens specialist. For correct use, it is important that you follow all the instructions given by your specialist and all the recommendations given on the packaging of your lenses and maintenance products.

INSTRUCTIONS:

Hard lenses are corneal lenses for daytime wear to correct short-sightedness, long-sightedness, astigmatism and presbyopia on regular or irregular corneas.

In children and young people under 20 years old, PRE AMYOPIC contact lenses can also slow down the progression of myopia.

RECOMMENDED RENEWAL:

Every 18 months

Every 12 months for contact lenses for children and young people.

PACKAGING:

The lenses are packaged in a mailer containing a decontaminant solution. It is fitted with a tamper-proof system until first opening. If packaging is damaged, do not use the lens.

The label provides the following information:

The lens brand name	
The name of its material	
The name and address of the manufacturer	
SN	The serial number
CE 0459	European conformity with health and safety requirements and certifying organisation number
	Expiry date
	Name of the solution preserver and expiry date
	Leaflet consultation
	not open if the packaging is damaged

PRECAUTIONS FOR USE

Your specialist will give you precise instructions on how to look after your lenses, put them in and take them out. Always wash and dry your hands thoroughly before putting your lenses in or taking them out. Never use damaged lenses. After use and before re-use, the lenses must be taken out, cleaned and decontaminated using the maintenance products your specialist has recommended. These lenses are designed to be worn by day; put them in the morning and take them out at night. The minimum recommended time for wearing PRE AMYOPIC lenses for them to be effective is 10 hours a day or more depending on physiological tolerance. Always take your lenses out before going to bed. If you do not wear your lenses for several days, refer to the information about your maintenance solution before re-using them. Always make sure you have your glasses with you (in case you find yourself obliged to take your lenses out). To avoid any alteration to the material or shape of your lenses, make sure you change them every 18 months or every 12 months for contact lenses for children and young people. Make sure that you attend the scheduled appointments with your practitioner; these are essential to monitor any changes.

PUTTING IN YOUR LENSES

Handling contact lenses entails a series of simple moves that should be done in respect of basic rules of hygiene.

- Before handling, hands must be washed, rinsed and dried on a clean, fluff-free towel.
- Put them in and take them out over a clean, flat, clear, well-lit surface.
- If you need to handle your lenses over a washbasin, make sure that the plug is in.
- To avoid any confusion between the right and left lens, it is best to always proceed in the same order: for example, first put the right lens in and then the left one.
- We recommend putting your lenses in after your morning wash and before putting make-up on; remove them after removing make-up and cleansing in the evening.
- **Place the lens on your index finger.**
- Use your middle fingers to open your eyelids
- Place the lens on the eye (figure 1). Repeat for the other eye.

TAKING OUT YOUR LENSES

First method

- Lean your head over a mirror placed flat on the table.
- Open your eyes very wide and stare at the mirror, slightly angling your head to the side of the eye concerned.
- Use your right index finger in the corner of your right eyelid and pull down at an angle, forcing yourself to keep your eyes wide open (figure 2)
- The lens will come unstick and fall onto the mirror.
- Repeat for the left lens.

Second method

- With your eyes wide open, place the index finger of one hand on the upper eyelid at the edge of the lashes and the index finger of the other hand on the lower eyelid, at the edge of the lashes. Now bring your two index fingers towards the centre, exerting gentle pressure on the edge of the eyelids to release the lens from the eye (figure 3).

MAINTENANCE

It is essential that you take care of your lenses. Always follow the instructions given on the maintenance products precisely. Decontaminate your lenses each time you take them out, to eliminate bacteria and be sure that you can continue wearing them comfortably, without risk.

Do not change the maintenance system your specialist has recommended.

Clean your container regularly, following the instructions given on the products used to maintain the lenses.

CONTRAINDICATIONS

Do not take any medications for eyes (eye drops or other ophthalmological products) without consulting your specialist.

Do not clean you lenses or container with tap water.

Do not use your saliva or any other household product to look after your lenses: only use the products recommended by your specialist.

Do not change brand of your maintenance product without consulting your specialist.

Do not allow your lenses to come into contact with lacquer or any other aerosol cosmetic product.

Do not leave your lenses in to sleep (see § precautions for use).

Do not wear your lenses in lying (having a bath or shower or if you are going to swim in a pool or the sea

Do not wear your lenses if exposed to dangerous fumes or irritants.

Pregnancy and some medications can cause tearing; talk to your specialist.

Wearing lenses is not recommended when you are suffering from an eye disorder and in very dry atmospheres (aeroplane, haidressers' hair-drying hood, etc.)

If you suffer from any of the following symptoms: irritated eye, redness, sensation of discomfort, watering eyes, a decline in sharpness of vision or poor sight, take the lens out and clean it. If discomfort persists, contact your specialist.

SIDE EFFECTS

The following possible side effects of lens wearing require you to stop wearing your lenses and consult a specialist:

- Irritated eye, discomfort, watering
- Vascularisation, abrasion, ulceration, abscess
- Photophobia

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